

STARTING RECORDING

3. The recorder starts searching for the capsule's signal. This message appears: "Clearing data..." and then "Waiting for pH capsule 1".
4. When the recorder identifies the capsule, this message appears: "Does this pH capsule ID number match the delivery device number?"
 - The capsule ID is printed on the packaging label. Bravo™ calibration-free reflux capsules have a 5-digit ID; Bravo™ reflux capsules that require calibration have a 4-digit ID. If the ID matches, press "Yes". If not, press "No" and the search begins again.
 - If the recorder still does not recognize the capsule or displays a capsule mismatch error message, repeat the procedure.
 - For two-capsule studies, repeat the above steps.

When capsule/recorder pairing is complete, the recorder automatically starts recording and the screen displays the symptom buttons.



STARTING RECORDING

5. Verify that the recorder is recording pH values and that the capsule status LED on the recorder is blinking.

TIP

There are 30 minutes to attach the capsule to the patient's esophagus before actual pH data starts to record for the configured study time duration.

